

# AED MAINTENANCE



CHECKUPS SHOULD BE COMPLETED MONTHLY  
OR BI-MONTHLY! THIS IS A SHORT CHECKLIST  
OF KEY FACTORS TO LOOK OVER:



- Note battery install by date and defib pad expirations when first setting up the AED.
- Check the status of the indicator light. It should show a flashing green light indicating ready to use status.
- Check and replace batteries if needed. Having backup batteries would also be great!
- Check for visible damage to pads and replace if needed/past expiration. Include backup pads.
- PPE supplies + more are included (gloves, razor, mask, trauma scissors, etc...)
- Check for visible damage on the machine. Ensure all buttons are working.

\*Look at your AED's administrator's guide for a more specific checklist!